

INSTRUCTION FOR USE

1. PRODUCT NAME: Sterile Hypodermic Needle for Single Use. (16G,18G,19G, 20G, 21G, 22G, 23G, 24G, 25G and 26G).

1.1 BRAND NAME: Nubeno

1.2 BASIC UDI: 8908017018HN014C

1.3 MATERIAL USED: Polypropylene, Adhesive (Epoxy Glue), SS 304.

1.4 INTENDED USE: - The Sterile hypodermic Needle is intended to be used with a luer slip or luer lock syringe for aspiration and injection of fluids for medical purpose.

1.5 INDICATIONS: - A hypodermic needle is used for rapid delivery of liquids, or when the injected substance cannot be ingested.

1.6 CONTRAINDICATIONS: -

- Device must not be used other than as indicated in intended use.

1.7 INSTRUCTION FOR USE: -

- Inspect package for integrity and expiry and then remove the needle from package.
- Be careful to pull the needle cap straight off to avoid the needle point damage.
- Push and twist the needle clock wise on the syringe tip to seal it firmly and ensure a tight fitting.
- Take or withdraw the medicinal fluids.
- Expel, air bubble through open end before injection.
- Device should be discarded after one procedure. It's extremely difficult to clean exactly after being expressed to biological materials and may causes adverse patient reactions if reuse, the cleaning of these devices may alter their structural property accordingly. Nubeno Healthcare will not be responsible for direct incidental and consecution damages resulting for reuse.

1.8 WARNINGS: -

- Device is single use only.
- Device must not be reused.
- Reuse of the device may cause serious infection, cross contamination, transmission of bacteria, viruses and blood related transfusion of serious diseases.
- Ensure before use that device/packing is not tempered or damage.
- Device for manual use only.

1.9 STORAGE: Store the devices at cool, dry and dark places.

NOTE: Avoid exposure to direct sunlight .it may lead to impairment of the needle's quality.

1.10 SHELF LIFE OF THE DEVICE: 05 Years from the manufacturing date.

1.11 DISPOSAL: - After use, dispose the device and packaging in accordance with hospital, administrative and/or local government Policy.

1.12 ADVERSE EVENT: The device is supplied with limited functionality, and there is no foreseeable risk associated with the device usage. Consequences from the improper use may cause or lead to redness, swelling, bruising etc.

For any feedback or query, customer/user can write or reach to us at:

Email- info@nubeno.in or Customer Care No: +91-0712-2284001.




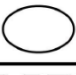


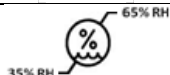
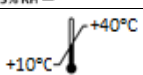
INSTRUCTION FOR USE



1.13 LABEL INFORMATIONS: -

Symbol	Description
R_x Only	Device to be sale on order/prescription of a Physician/ Doctor/ Paramedical staff only
REF	Catalogue Number
LOT	Lot Number: Batch Number
	Expiry date: Device can be used until the end of the month indicated
STERILE EO	Sterilization with ethylene oxide gas
	Single use only
	Manufacturer
	Warnings
CE 0123	Conformité Européenne (CE marking is a mandatory conformity marking for certain devices sold within the European Economic Area)
	Do not Re-sterilize
	Non-Pyrogenic
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight
	Date of manufacturing
	Consult Instruction for Use
	Safe disposal

INSTRUCTION FOR USE

	Do not use hook
	Store and stack in upright condition
	Unique device identifier
	Single sterile barrier system (For primary package only)
	Medical device
	Country of manufacture
	Moisture limitation
	Temperature limitation



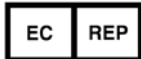
NUBENO HEALTHCARE PRIVATE LIMITED

Plot No. 10A Sector 23, Mihan Nagpur, Maharashtra India (441108)

Mfg. Lic. No. MFG/MD/2021/000002

E-mail: info@nubeno.in, **Customer care:** +91 0712 – 2284001

Regd. Office: Nava Bharat Bhavan Chhatrapati Square, Wardha Road Nagpur, Maharashtra – 440010



MED DEVICES LIFESCIENCES B.V.

Add.: Keizersgracht 482, 1017 EG Amsterdam, The Netherlands

E-mail: info@meddevices.net

Phone: +31-202254558