

INSTRUCTION FOR USE

- 1. PRODUCT NAME: Sterile Luer Lock Hypodermic Single Use Syringe With/Without Needles**
(1mL, 2mL, 2.5mL 3mL, 5mL, 10mL, 20mL) with needle sizes (16G, 18G, 19G, 20G, 21G, 22G, 23G, 24G, 25G, 26G,
 - 1.1 BASIC UDI:** 8908017018LLSY01GC
 - 1.2 TRADE NAME:** Nulok
 - 1.3 MATERIAL USED:** - Polypropylene, Thermoplastic Elastomer, Stainless Steel (304).
 - 1.4 DEVICE DESCRIPTION:** A syringe is a simply consisting of a plunger/piston & gasket that's fits tightly within a cylindrical tube called a barrel. The plunger can be linearly pulled and pushed along the inside of the tube/barrel, allowing the syringe to take-in and expel liquid or gas through a discharge orifice/nozzle/barrel tip at the front (open) end of the tube/barrel. The open end of the syringe may be fitted with a hypodermic needle, a luer lock nozzle or tubing to help direct the flow into and out of the barrel.
 - 1.5 INTENDED USE:** - The Product is intended to be used for medical purposes to inject fluids into or withdraw fluids from the body.
 - 1.6 INDICATION FOR USE:** - Sterile Luer Lock hypodermic syringe to be used for medical purpose to inject fluids into or withdraw fluids from the body.
 - 1.7 INTENDED USER:** The medical device shall be used by qualified trained medical staff.
 - 1.8 PATIENT POPULATION:** Sterile Luer Lock hypodermic syringe shall be used by qualified and specifically trained medical staff. The selection of hypodermic Syringe & Needle for usage on patients is dependent on combination of patient age, physical condition and medication requirements.
 - 1.9 CONTRAINDICATIONS:** -
 - Device must not be used other than as indicated in intended use.
 - Device should not be used for administration of blood and blood components.
 - Device should not be used for large volume fluids and fluids derivatives.
 - Device should not be used for administration of high viscous fluids.
 - Device should not be used on patients with known hypersensitive to device material.
 - 1.10 INSTRUCTION FOR USE:** -
 - Inspect package for integrity and expiry and then remove the syringe from package.
 - Be careful to pull the needle cap straight off to avoid the needle point damage.
 - Push and twist the needle clock wise on the syringe tip to seal it firmly and ensure a tight fitting.
 - Attach the needle to the luer lock tip of the syringe by twisting it on. Make sure it's securely attached.
 - Take or withdraw the medicinal fluids.
 - Expel, air bubble through open end before injection.
 - Nubeno Healthcare will not be responsible for direct incidental and consecution damages resulting for reuse.

INSTRUCTION FOR USE

1.11 WARNINGS/PRECAUTIONS: -

- Device must not be reused. Reuse of the device may cause serious infection, cross contamination, transmission of bacteria, viruses and blood related transfusion of serious diseases.
- Do Not Resterilise.
- Ensure before use that device/packing is not tempered or damage.
- Device for manual use only.

1.12 DISCLAIMER:

- Device should not be used out of its indications/instructions for use. In-case the device is used out of its instructions described, the manufacturer/supplier must be informed for any usage outside of its recommendations.

1.13 STORAGE OF THE DEVICE: - Store in cool and dry place & avoid exposure to direct sunlight

1.14 SHELF LIFE OF THE DEVICE: 05 Years from the manufacturing date.










1.15 DISPOSAL METHOD: - Used medical device is considered as biohazard waste dispose-off the used medical device in accordance with hospital, administrative and/or local government policy.

1.16 ADVERSE EVENT: The device is supplied with limited functionality, and there is no foreseeable risk associated with the device usage. Consequences from the improper use may cause or lead to redness, swelling, bruising etc.













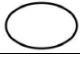



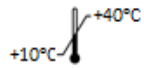
1.17 For any feedback or query, customer/user can write or reach to us at:

Email- info@nubeno.in or Customer Care No: +91-0712-2284001

1.18 LABEL INFORMATIONS: -

Symbol	Description
	Device to be sale on order/prescription of a Physician/ Doctor/ Paramedical staff only
	Catalogue Number
	Lot Number: Batch Number
	Expiry date: Device can be used until the end of the month indicated
	Sterilization with ethylene oxide gas
	Single use only
	Manufacturer
	Warnings
	Conformité Européenne (CE marking is a mandatory conformity marking for certain devices sold within the European Economic Area)

INSTRUCTION FOR USE

	Do not Re-sterilize
	Non-Pyrogenic
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight
	Date of manufacturing
	Consult Instruction for Use
	Safe disposal of syringe
	Do not use hook
	Store and stack in upright condition
	Fragile, handle with care
	Unique device identifier
	Single sterile barrier system (For primary package only)
	Medical device
	Country of manufacture
	Moisture limitation
	Temperature limitation



NUBENO HEALTHCARE PRIVATE LIMITED

Plot No. 10A Sector 23, Mihan Nagpur, Maharashtra India (441108)

Mfg. Lic. No. MFG/MD/2021/000002

E-mail: info@nubeno.in, Customer care: +91 0712 – 2284001

Regd. Office: Nava Bharat Bhavan Chhatrapati Square, Wardha Road Nagpur, Maharashtra – 440010



MED DEVICES LIFESCIENCES B.V.

Add.: Keizersgracht 482, 1017 EG Amsterdam, The Netherlands

E-mail: info@meddevices.net

Phone: +31-202254558